



RX Do's & Don'ts

Do's:

- Only use edgers that are designed for wrapped lenses.
- Wrap RX compensation is done on all Wiley X, Inc. frames.
- Only use polycarbonate material lenses in a Z87 stamped frame.
- All lenses must be cold snapped into Wiley X, Inc. frames.
- When in doubt, please call the Wiley X RX Dept. at 1.888.210.8926

Because of the specific lens requirements in our ANSI certified frames we recommend that you only use an authorized* Wiley X Lab for filling prescriptions. Filling a prescription within an authentic Wiley X, Inc. frame through a non-authorized lab will void your frame warranty. Please contact your Wiley X, Inc. representative for a list of authorized labs.

Don'ts:

- Never heat a Wiley X, Inc. frame.
- Do not remove the information cards that come in the case with your Wiley X, Inc. frames. There is crucial information for outside labs in order to fill prescriptions within an authentic Wiley X frames to meet required ANSI standards.
- Do not cut RX lenses oversize; this will stress the frame causing it to crack over time.
- Do not put a flat lens into a wrapped frame, without the proper bevel equipment as this can splay the temples rendering it unusable.

Here at Wiley X, Inc. we know how important prescription eyewear is to you and your customer, therefore all Wiley X, Inc. eyewear has been compensated accordingly for wrap, tilt, and vertex distance to achieve superior optical quality. This may result in slight RX power differential between the RX lenses and the one you have prescribed to custom fit specific vision needs.

ENJOY YOUR WILEY X EYEWEAR!!

*Placing Rx orders through Wiley X authorized labs will maintain the Wiley X frame warranty. However, Wiley X cannot be held responsible for Rx orders placed by dealers directly through authorized labs. Wiley X will be responsible for Rx orders placed through the Wiley X Rx Department only.